

A NOVEL METHOD FOR URINE DRUG SCREENING BASED ON DESORPTION ELECTROSPRAY IONIZATION (DESI) MS ANALYSIS

Julia Balog¹, Erika Stark¹, Eltahir I Elbakri², Rav Sheth¹, Jon Danaceau¹, Praveen Kumar¹, John Vukovic¹, Steven Pringle³, Patrice Ohouo²

¹Waters Corporation, Milford, MA, ²CleanSlate Centers, Inc. and Total Wellness Centers, Holyoke, MA, ³Waters Corporation, Wilmslow, UK.


The goal of this study was to develop an ultra-rapid mass spectrometry-based screening method capable of routine detection of drug metabolites in urine samples.

Drug of abuse screening – Can we use a fast mass spectrometry-based screening method instead of immunoassays?

- All samples undergo screening
- Automated, cheap, rapid response time for the requesting clinicians
- Selective detection of illicit and therapeutic agents across many drug classes
- Higher specificity than conventional immunoassay-based screening
- Decreased false positive/negative results
- Flexibility and improved capacity to test for Novel Psychoactive Substances (NPS)

DESI TECHNOLOGY

- A charged solvent is directed towards the sample mobilizing molecules from the sample surface
- Directly analyzed by MS – ambient ionization technology
- 1-2 seconds per sample

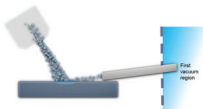

- Prototype DESI XS with automated slide sorting capability
- 20 plates (80 slides)
- Using 384 well plates with 3s per sample = 6.5 hours analysis time ~7500 patient samples

DRUG PANEL


Is DESI Sensitive enough: Cutoff concentrations between 2-100ng/mL, need to quantify 40% of cutoff (0.8–40 ng/mL)

General Class	Compound	MS ion Mode	Transition 1	Cutoffs (ng/mL)	40% LoQ (ng/mL)
Fentanyl	Fentanyl	ES+	337.2 > 105.1	2	0.8
Fentanyl	Fentanyl, nor	ES+	233.1 > 84.1	2	0.8
Buprenorphine	Buprenorphine	ES+	468.2 > 84.1	5	2
Buprenorphine	Buprenorphine, nor	ES+	414.2 > 83.1	5	2
Opiates/ Opioids	Morphine, 6-Acetyl	ES+	328.1 > 165.1	5	2
Cannabinoids	(-)-Δ9-THC	ES (-)	213.3 > 193.2	5	2
Creatinine	Creatinine (mg/dL)	ES+	114.0 > 86.0	20	8
Gabapentinoid	Gabapentin	ES+	173.1 > 155.2	500	200
Cannabinoids	(-)-11-nor-9-Carboxy-Δ9-THC	ES (-)	343.2 > 299.2	5	2
Benzodiazepines	Alprazolam	ES+	309.1 > 281.1	100	40
Benzodiazepines	alprazolam, α-Hydroxy	ES+	325.1 > 297.1	100	40
Antidepressants	Amitriptyline	ES+	278.2 > 91	100	40
Amphetamines	Amphetamine	ES+	136.1 > 119.1	100	40
Carisoprodol	Carisoprodol	ES+	261.2 > 176.1	100	40
Benzodiazepines	Clonazepam, 7-Amino	ES+	286.1 > 121.1	100	40
Opiates/ Opioids	Cocaine	ES+	300 > 215.1	100	40
Antidepressants	Cyclobenzaprine	ES+	276.2 > 215.2	100	40
Dextrophan/ Levorphanol	Dextrophan	ES+	258.2 > 133.1	100	40
Benzodiazepines	Diazepam, nor	ES+	271.1 > 140.1	100	40
Methadone	EDDP	ES+	278.2 > 234.2	100	40
Benzodiazepines	Flurazepam, 2-Hydroxyethyl	ES+	333.1 > 109.	100	40
Opiates/ Opioids	Hydrocodone	ES+	300.1 > 199.1	100	40
Opiates/ Opioids	Hydrocodone, nor	ES+	286.1 > 199.1	100	40
Opiates/ Opioids	Hydromorphone	ES+	286.1 > 185.1	100	40

General Class	Compound	MS ion Mode	Transition 1	Cutoffs (ng/mL)	40% LoQ (ng/mL)
Benzodiazepines	Lorazepam	ES+	321 > 275.1	100	40
Amphetamines	MDA	ES+	180.1 > 163.1	100	40
Amphetamines	MDMA	ES+	194.1 > 163.1	100	40
Carisoprodol	Meprobamate	ES+	219.1 > 158.1	100	40
Methadone	Methadone	ES+	310.2 > 105.1	100	40
Amphetamines	Methamphetamine	ES+	150.1 > 119.1	100	40
Opiates/ Opioids	Morphine	ES+	286.1 > 201.1	100	40
Ketamine	Norketamine	ES+	224.1 > 125	100	40
Antidepressants	Nortriptyline	ES+	264.2 > 105	100	40
Benzodiazepines	Oxazepam	ES+	287.1 > 241.1	100	40
Opiates/ Opioids	Oxycodone	ES+	316.1 > 241.1	100	40
Opiates/ Opioids	Oxycodone, nor	ES+	302.1 > 227.1	100	40
Opiates/ Opioids	Oxymorphone	ES+	302.1 > 227.1	100	40
Gabapentinoid	Pregabalin	ES+	160.1 > 97.1	100	40
Opiates/ Opioids	Tapentadol	ES+	222.2 > 107.1	100	40
Benzodiazepines	Temazepam	ES+	301.0 > 255.1	100	40
Tramadol	Tramadol	ES+	264.1 > 58.1	100	40
Tramadol	Tramadol, n-desmethyl	ES+	250.2 > 44.1	100	40
Benzodiazepines	Triazolam, α-Hydroxy	ES+	359.1 > 176.1	100	40
Cocaine	Benzoylcegonine	ES+	290.2 > 168.1	100	40
Cocaine/ alcohol	Cocacethylene	ES+	318.2 > 196.1	100	40
Sedative/ Z drugs	Zolpidem phenyl-4-carboxylic acid	ES+	338.2 > 92.1	100	40
Kratom	Mitragynine	ES+	399.2 > 174.1	20	8
Alcohol	Ethyl sulfate (EIS)	ES (-)	124.9 > 97	250	100

SAMPLE PREPARATION

In addition to dissolving in solvent two preparation methods were used (dilute and shoot and SPE) before spotting 1uL onto the sample plate

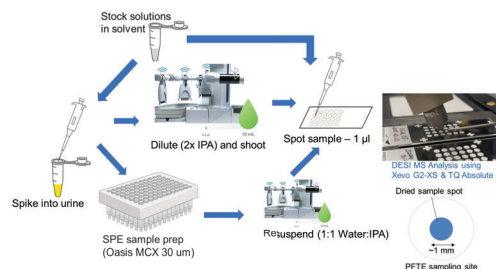


XEVO™ G2-XS TOF MS DATA

Dextrophan

• DESI detection limit (LoD)

- Dilute and shoot: 40% LoQ in solvent
- Dilute and shoot: 500% LoQ in urine
- SPE sample prep: 40% LoQ in urine
- *LoQ (Limit of quantitation) = Threshold or cutoff

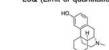

[M+H]⁺ = 258.185

Concentration for 100% LoQ is 100 ng/mL

Concentration for 100% LoQ is 100 ng/mL

Fentanyl

• DESI detection limit:

- Dilute and shoot: 40% LoQ in solvent
- Dilute and shoot: >500% LoQ in urine
- SPE: 100% LoQ in urine



[M+H]⁺ = 337.227

Concentration for 100% LoQ is 2 ng/mL

XEVO TQ ABSOLUTE MS DATA

We can detect 40% of the cutoff value in urine

Q-TOF

Full spectrum

Fentanyl - QTOF

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227

337.227


337.227

337.227

337.227

337.227

337.227

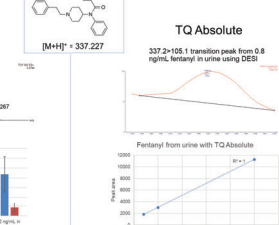

OVERALL PERFORMANCE

Number of drug molecules detected from urine with different methods (n=45)

	Xevo G2-XS no SPE	Xevo TQ Abs SPE
40% LoQ	6	25
100%LoQ	16	35
500%LoQ	30	41


• Molecules not detected at 100% LoQ:

- Fentanyl, nor (2 ng/mL)
- Buprenorphine, Buprenorphine, nor, 6-MAM (5 ng/mL)
- Pregabalin, MDA (100 ng/mL)

TOWARDS QUANTITATION


• Measuring Buprenorphine concentration vs Buprenorphine-d4 standard

- BUP-d4 concentration 100ng/mL
- BUP concentration: 10, 25, 50, 100, 250, 500 ng/mL
- Area under the curve

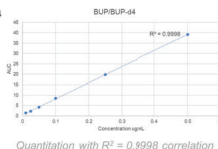

ANOMALY DETECTION

• Using Principal Component Analysis (PCA) and Linear Discriminant Analysis (LDA) for the detection of 'outlier samples'

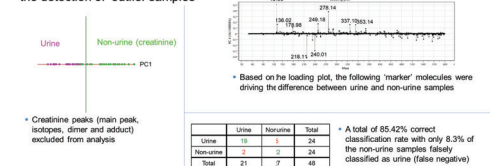

CONCLUSIONS

- Drug of abuse molecules ionize well with DESI – that is mandatory if we want to push the sensitivity limits
- We have created an automated, flexible, rapid plate reading system using DESI technology with options to run 80 slides or 20 plates without supervision and export data on the fly
- Using some specific sample preparation, we can reach the desired sensitivity for most of the molecules with DESI (2 ng/mL >100 ng/mL) depending on the molecule
- Using parallel TOF and MRM methods allow us to:
 - Spot anomalies regarding urine or non-expected molecules
 - Immediately include novel psychoactive substances in the detection method once identified

For Research Use Only

Waters & Xevo are trademarks of Waters Technologies Corporation. CleanSlate is a trademark of CleanSlate Centers, Inc.

TO DOWNLOAD A COPY OF THIS POSTER, VISIT WWW.WATERS.COM/POSTERS

720009085EN
©2024 Waters Corporation